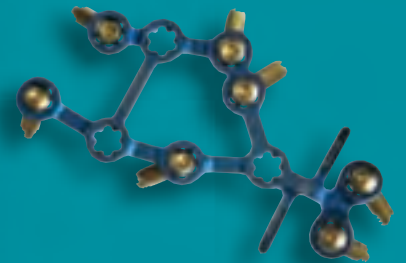
VARIABEL WINKELSTABILE

CALCANEUSPLATTE

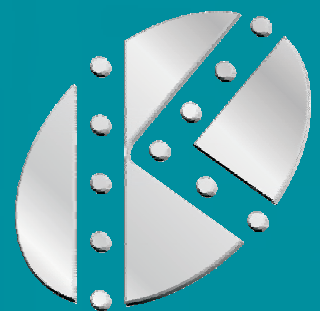
Variable-angle-stable calcaneusplate

- Titan

KLEINFRAGMENT

Small fragment

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The management of intra-articular calcaneal fractures continues to be a challenge. Apart from the complex anatomy and fracture pathology, surgical success is limited by soft tissue problems and the relative rarity of the fracture. However, the prognosis depends to a crucial degree on the quality of the operative management. Besides anatomical knowledge and a good understanding of fracture forms, an implant is therefore required which reliably fixes the fragment position achieved by reduction, possesses as many degrees of freedom as possible and avoids soft tissue irritation. The principle of variable angle-stable screw fixation improves the fixation possibilities considerably, particularly in multifragment or comminuted situations. The following summarised instructions on indications, preoperative phase, operation technique and suggestions for follow-up treatment are intended to be an aid to mastering the problems particularly of higher-grade intra-articular calcaneus fractures.

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- CT necessary for therapy planning
- initial closed rough reduction, if necessary with retention in external fixator, reduces soft tissue damage due to internal fracture pressure and prevents soft tissue shrinkage
- operation time possible within the 6h limit, but usually around the 7th day (possible up to 3 weeks after trauma)
- look for concomitant injuries

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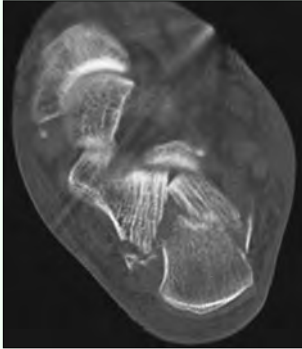
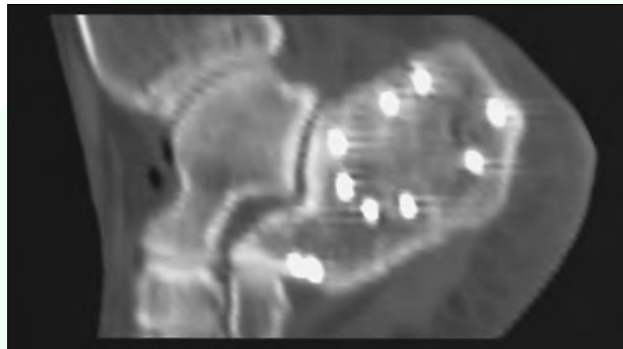
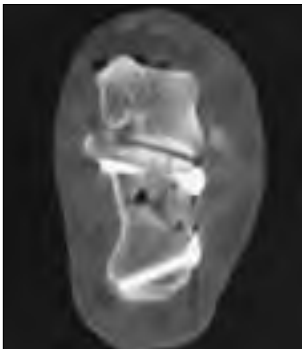
- no plaster
- movement exercises incl. motor movement splint for ankle and subtalar joint
- mobilisation with sole contact or relieving orthosis
- no limiting of extent of movement
- long-soled hard shell inlays if necessary
- full loading after 10-12 weeks
- no routine implant removal but can be done as part of arthrolysis

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Lateral or face-down position (with the latter, combined medial procedure is simplified and it is easier to bring out the K-wires), antibiotic, tourniquet (temporary), extended lateral approach

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Exposure of the fracture laterally, Retraction of the soft tissue flap using sheathed K wires (protect soft tissues!)

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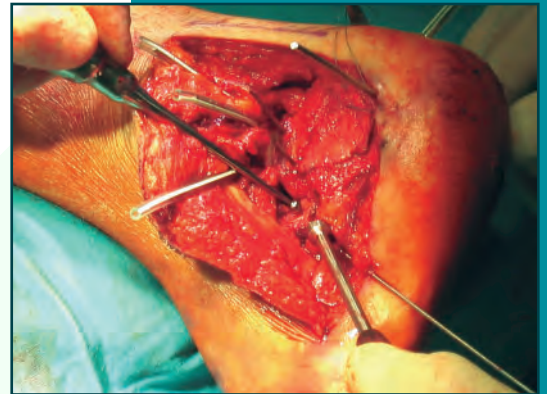
Fold back the lateral bulge and reconstruct the posterior joint facet from medial to lateral, using the (usually) undislocated sustentaculum fragment for orientation
Check for correct position using a palpating hook, open arthroscopy, intraop. 3D radiography or purely visually.

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
Reduction of the tuberosity fragment (correcting shortening, varus, malrotation and subluxation as necessary), if necessary reduction of the anterior process fragments and finally of the lat. bulge

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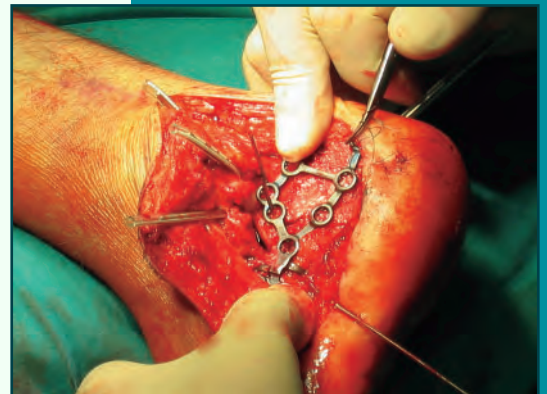
Depending on the fracture situation, place angle-stable or tension screw(s) in the subtalar plate holes and in the remaining holes

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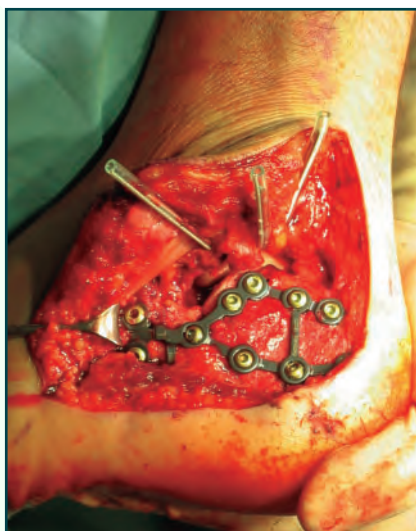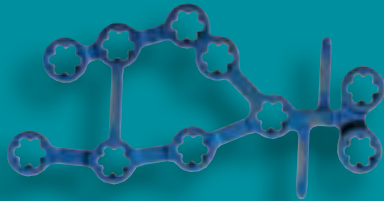
Cancellous screws, D 4,0 mm (conical head thread, self-tapping)

Bestell-Nr. Code N°	Länge Length	Anzahl Quantity
3.147.22	22 mm	5
3.147.26	26 mm	5
3.147.30	30 mm	5
3.147.35	35 mm	5
3.147.40	40 mm	5
3.147.45	45 mm	5
3.147.50	50 mm	5

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Cortical screws, diameter 3,5 mm, conical head thread (self-tapping)

Bestell-Nr. Code N°	Länge Length	Anzahl Quantity
3.133.22	22 mm	5
3.133.26	26 mm	5
3.133.30	30 mm	5
3.133.35	35 mm	5
3.133.40	40 mm	5
3.133.45	45 mm	5
3.133.50	50 mm	5

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Cancellous screws, diameter 4,0 mm, fully threaded

Bestell-Nr. Code N°	Länge Length	Anzahl Quantity
3.145.22	22 mm	3
3.145.26	26 mm	3
3.145.30	30 mm	3
3.145.35	35 mm	3
3.145.40	40 mm	3
3.145.45	45 mm	3
3.145.50	50 mm	3

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

Cannulated screw, D 4,0 mm (short thread, self-drilling)

Bestell-Nr. Code N°	Länge Length	Anzahl Quantity
3.440.40	40 mm	3
3.440.42	42 mm	3
3.440.44	44 mm	3
3.440.46	46 mm	3
3.440.48	48 mm	3
3.440.50	50 mm	3
3.440.55	55 mm	3

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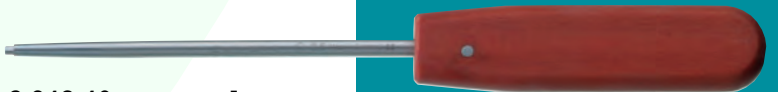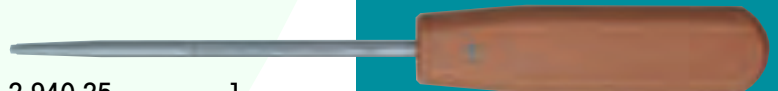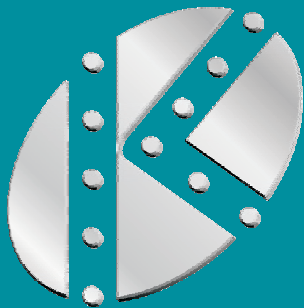
No part of this publication may be reproduced, stored in a retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying, recording or otherwise without the prior written permission of the publisher.

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© Copyright Königsee Implantate GmbH
OSTEOSYNTHESE GmbH
ZUR OSTEOSYNTHESE GmbH**